

# bruckhoff

la belle  
air conduction

junior  
bone conduction

Operating manual

- EN -

la belle  
bone conduction

headset  
bone conduction



0123

Dealer's stamp

B17034 - 8/17

# bruckhoff

AUDIA AKUSTIK GMBH

Herrenstraße 6

30159 Hannover

Germany

Tel: +49 (0) 511 16700-0

E-Mail: [service@bruckhoff.com](mailto:service@bruckhoff.com)

[www.bruckhoff.com](http://www.bruckhoff.com)



We understand hearing.

Turning the unit on/off

la belle

la belle

junior

headset

### Identical for all control units:

When the battery cover is closed, your hearing aid is always switched on automatically.

The hearing aid is turned off by simply opening the battery cover.



Battery cover closed



switched on

Battery cover open



switched off

Changing the battery

la belle

la belle

junior

headset

### Identical for all control units:

Open the battery cover. Remove the battery with the supplied magnetic pen.

Insertion is in reverse order:

Insert the new type 312 battery with the „+“ sign facing up. Close the cover. The hearing aid is automatically switched on again!

Magnetic pen



Battery  
Type 312 standard\*



\* Optional:  
Battery  
Type 13



Safety, care and use information. Please read before use!



**Your hearing aid does not restore normal hearing, nor does it prevent or improve an organically induced hearing loss.**

### Safety and Warnings

- Your hearing aid must not be worn by other persons. A risk of hearing damage will ensue.
- Unauthorized modifications to the hearing aid may damage your hearing.
- Never wear your hearing aid in areas where there is a risk of explosion.
- Keep the hearing aid out of the reach of small children, people with cognitive impairment and pets. The ingestion of hearing aid parts, in particular of batteries, is hazardous to life. There is a danger of suffocation. In case of emergency call a doctor or a hospital immediately.
- If you are experiencing pain in or behind the ear, or if you develop increased earwax, inflamed ear canals or otherwise exposed mid-ear cavities. In these cases, we recommend the use of a custom ear mould. In the rather unlikely event that the dome remains in the au-

ditory canal, we recommend a visit to a specialist medical practitioner.

Listening programs in the directional microphone mode reduce background noises. Warning signals or noises coming from behind are not perceived or only weakly perceived.

Never connect your hearing aid to an external audio device such as a radio, etc., as this may result in physical injury (electric shock).

The use of a hearing aid by persons with an actively implanted medical device (for example heart pacemaker, defibrillator, etc.) requires prior consultation and clarification with a specialist physician. (For further information please refer to our website at [www.bruckhoff.com/Sicherheitshinweise](http://www.bruckhoff.com/Sicherheitshinweise))

### Information on product safety, use and maintenance

Your hearing aid is very robust and can be worn during all normal everyday activities, even under extreme conditions for short periods.

- Never immerse it in/under water or use it while showering, bathing or swimming. The hearing aid electronics will become damaged.
- Never clean the microphone openings

with moist agents. They will lose their special acoustic properties.

- Protect your hearing aid from heat.
- Never use a microwave or other heating appliances to dry it.

The dome must be replaced every three months, or as soon as it becomes hard or brittle. This prevents unintentional detachment of the dome during insertion and removal from the ear.

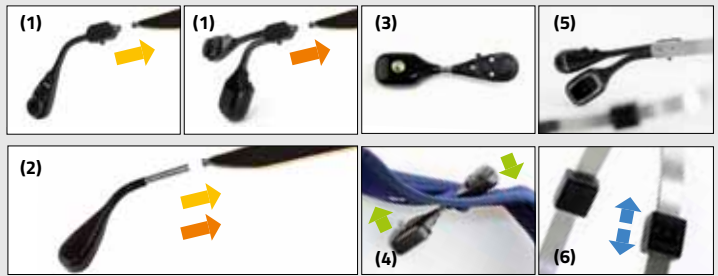
- As far as possible, leave unused hearing aids with an open battery compartment so that moisture can escape.
- For long-term non-use, remove the battery.
- Always store the hearing aid in a safe, dry, and clean place
- Do not drop it!
- Always replace weak or defective batteries (e.g. liquid loss) immediately. Always use new batteries
- Do not dispose of old batteries in domestic waste. Your hearing care professional will dispose of these for you.
- Use only batteries with max. 1.5 Volt.
- Never use silver-zinc or lithium-ion batteries.
- Some (dental) medical examinations with x-rays, e.g. CT examination, MRT/NMRT, can impair the functioning of your hearing aid. Remove your hearing aid beforehand.

- Hearing aids do not have to be removed before passing through a security sluice (e.g. at the airport). The low X-ray radiation emitted there does not affect your hearing aid.
- Do not use your hearing aid in locations where electronic devices are prohibited.

Detailed information can be found on the internet at [www.bruckhoff.com/Sicherheitshinweise](http://www.bruckhoff.com/Sicherheitshinweise) In case of doubt, please contact your hearing healthcare professional or a specialist physician.

**Fitting and wearing instructions**

Insert the la belle listening module (1) on your spectacle arm. Use your glasses without hearing aid - using the push-on spectacle end piece (2). The children's junior hearing system (3) is connected to the headband using two snaps (4). (5) The headset hearing system can be easily adjusted to your head shape via the pull-out adjustment (6).

**Setting options on the control unit 1**

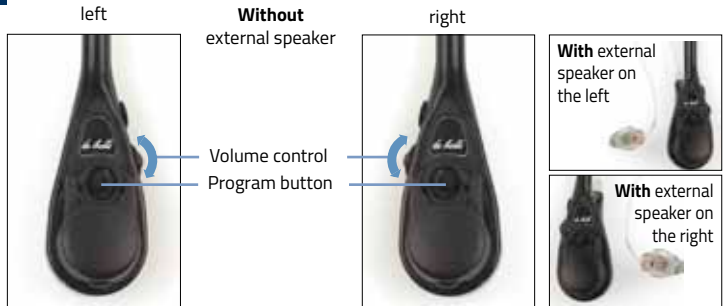
There are 5 control unit versions, each of which differs slightly in its operability.

**Air conduction**

- la belle (with/without external microphone)

You can set the following functions yourself:

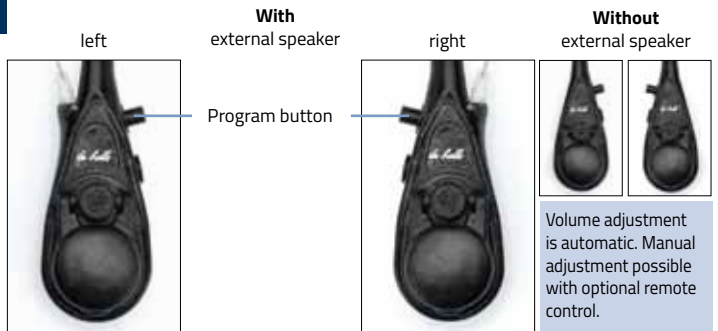
- Adjust the volume
- Select the listening programs
- Switch the unit on/off: see the instructions on the other side

**Setting options on the control unit 2****Air conduction**

- la belle (with/without external microphone)

You can set the following functions yourself:

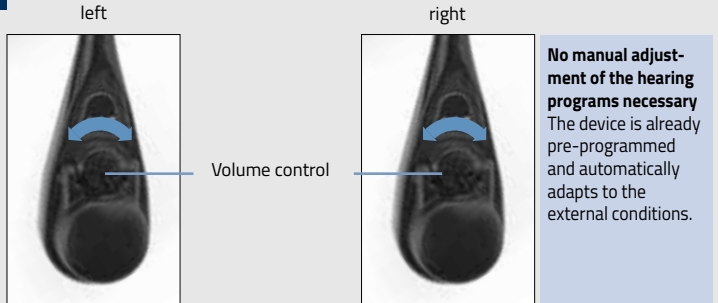
- Select the listening programs
- Optional: volume control with remote control
- Switch the unit on/off: see the instructions on the other side

**Setting options on the control unit 3****Bone conduction**

- la belle (control unit above/below)

You can set the following functions yourself:

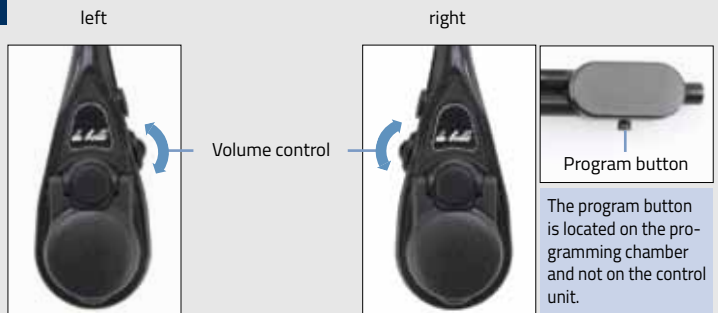
- Adjust the volume
- Switch the unit on/off: see the instructions on the other side

**Setting options on the control unit 4****Bone conduction**

- la belle (control unit above/below)
- junior
- headset

You can set the following functions yourself:

- Adjust the volume
- Select the listening programs
- Switch the unit on/off: see the instructions on the other side

**Setting options on the control unit 5****Bone conduction**

- junior

You can set the following functions yourself:

- Select the listening programs
- Adjust the volume
- Switch the unit on/off: see the instructions on the other side

